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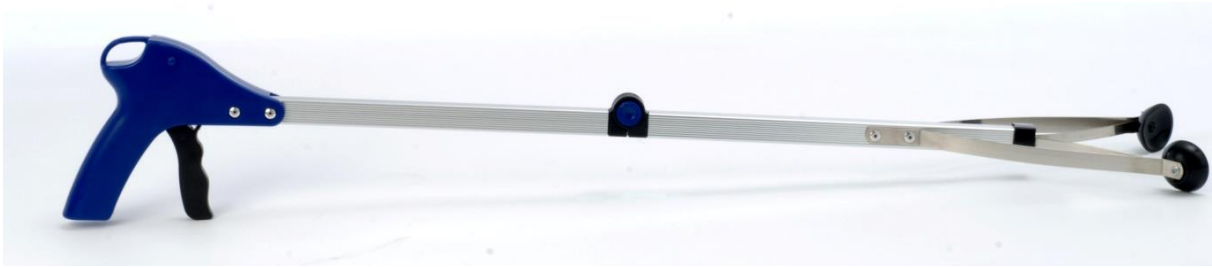
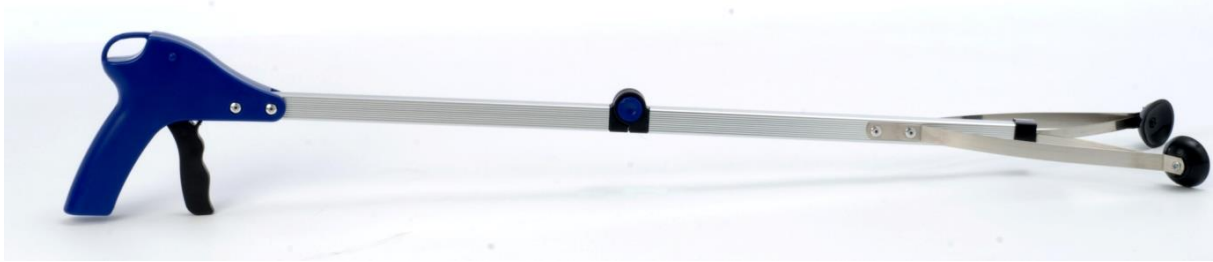
Reacher RFM with Suction Tip, foldable

User Manual

EN

Purpose

The reacher assists the user in picking up, holding, and gripping smaller objects that weigh no more than 1 kg.

Application

1. Separate the reacher from the cardboard.
2. Unfold the reacher and secure it with the snap button in the middle. Do not use the reacher if it cannot be securely fixed!
3. Now, by pulling the handle together, you can close the grippers and lift lightweight objects with them.
4. If you want to keep the gripper in position and relieve your hand, you can fix the grippers by pulling back the slider on the left side of the reacher. By pushing the slider forward (while the handle is engaged), the rigidity is released, and the reacher can be used as before.
5. To fold it back, release the snap button in the middle and fold the reacher together.

Safety Instructions

- Use the foldable Reacher RFM with Suction Tip only for lightweight objects up to a maximum of 1 kg.
- When pulling the handle together, be careful not to trap clothing or fingers.
- The reacher is not suitable for use in personal hygiene and food preparation.
- Check the functionality of the tool before use.
- Do not use a defective reacher.

Cleaning

Clean the surfaces of the reacher with a pH-neutral cleaning agent. Please observe the following instructions:

- Do not use abrasive agents, corrosive substances, acids, bleach, or hard cleaning tools, as well as high-pressure cleaners.
- The item should be wiped down with a conventional disinfectant according to the respective package instructions. Disinfection is mandatory before passing it on to another user!

Scope of Delivery

Reacher RFM with Suction Tip, foldable- ready for use
User manual

Technical Data

Material: Aluminum, plastic (polyethylene)
Length: 81 cm, folded 42 cm
Weight: 210 g
Load capacity: approx. 1 kg

Warranty

The warranty period is 24 months and covers all product defects that can be attributed to manufacturing or material defects.

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Rehaforum Medical GmbH, Daimlerstraße 12a, DE-25337 Elmshorn