AT51111

ALUMINIUM PREMIUM ROLLATOR Produktspezifikation Introductory Information

EINFÜHRENDE INFORMATIONEN

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Introductory information

Thank you for your decision to use our Aluminum premium Rollator. The quality will not disappoint you. Please read the Instruction manual carefully before you use the Rollator for the first time. The Instruction includes important safety devices and

Value tips for the correct authorization and care of the rollator. If you have any questions or need information, please contact the specialist dealer who delivered you the Aluminum Premium Rollator. He will be happy to advise you. The operating instructions can also be viewed electronically and, in a more regular form, can also be used by people with poor eyesight.

General

The instruction manual given to you is intended to help you to improve the operation and handling of the rollator. This is a festive and important component for the use of the rollator. Keep the instructions to hand and give them over when passing on the rollator. Please read the Instruction manual carefully before you use the Rollator for the first time.

Characteristics

The Aluminum rollator was designed for indoor and outdoor use. It is meant as a walking aid with only one seat (1 person). Carriage of more than one person is not allowed. Unless otherwise noted, the operating instructions are aimed at the patient rather than the user of the Walker.

Contraindications

The use of the rollator is unsuitable for:

- · cognitive disorders
- · severe balance disorders
- · lack of possibility to use the arms
- · incapacity to sit

Durability

The expected service life of the rollator is up to a maximum of 5 years if used as intended and if the safety, care and maintenance instructions are observed. The rollator can continue to be used beyond this period, provided it is in good working order.

Responsibility

As a manufacturer, we can only consider ourselves responsible for the safety, reliability and usability of the rollator if any changes, extensions, repairs and maintenance work are only carried out by those persons who have been authorized by us and the rollator is used in compliance with all instructions for use.

EC declaration of conformity

As the manufacturer of the product, Medical Medizin- und Rehabilitation products GmbH declares that the Aluminum Premium Rollator meets the requirements of the European Directive 93/42/EEC.

Symbole

Important! Designates particularly safety-relevant notes. Failure to follow notices may result in personal injury or damage to the rollator.!

Important! Designates very useful information.

Observe the instructions for use

CE-Marking

CLASSIC Pocket Rollator Overview

- 1. Backrest
- 2. Bag
- 3. Bracket of the front wheels
- 4. Front wheels (Controllable)
- 5. Adjustment screw brake
- 6. Rear
- 7. Tilting aids (Left and Right)
- 8. Bracket of rear wheels
- 9. Seat
- 10. Height adjustment of the handles
- 11. Brakelever
- 12. Sliding handle

Setting up the Classic Pocket

When installing the rollator, you can be assisted by your specialist dealer. If you have manual skills, you can carry out the assembly on your own.

When installing the rollator, you should consider the risk of clamping!

Handle height

You can adjust the height of the handles to your height. You don't need a tool to do this.

Below each handle you will find a red toggle switch (Fig. 3). Press and hold the toggle switch. Now you can pull up or push the handle down. Release the button. Now drag or press again until you hear and notice that the handle snaps into place and can no longer be adjusted. Repeat on the other side to set both handles to the same height.

Fig. 3

Bag

The rollator comes with mounted bag. The bag can be easily removed or hung over the guide rails (Fig. 4).

Fig. 4

Brakes

The adjustment of brakes should be carried out by trained professionals. However, if you have sufficient experience, you can do this work yourself. Keep in mind, however, that improperly adjusted brakes can have lifethreatening consequences.!

Strengthen/reduce braking effect

- 1. Loosen the Inbus screw.
- 2. Bowden pull retightening or loosening by pulling out or pushing it out (Fig. 5 arrow 2).
- 3. Retighten the Inbus screw.
- 4. Fine adjustment:

Retract protective cap (Fig.6, arrow 1) until the adjustment screw can be rotated (Fig.6 arrow 2). Turn the adjustment screw clockwise: increase braking effect. Push the protective cap back and tighten it.

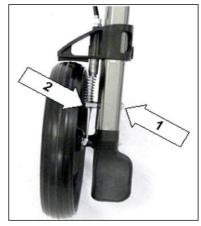
Fig. 5

Perform these steps on the other brake as well. The brakes must be adjusted in such a way that they do not cause resistance during movement, but allow easy progress. Tightened brakes make it impossible to use the rollator.

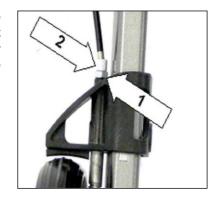
Fig.6

Using the Classic Pocket

Especially when folding and unfolding the rollator as well as when preparing for transport, the risk of clamping must be taken into account!

Folding and unfolding the rollator

The rollator can be stored in a folded state in a space-saving manner. To use it again, press down the seat on the outside (Fig.7). There is a lock on the underside of the rollator to secure it against accidental folding. If you hear a clear slip-in when folding, the locking is properly locked. To fold, pull up the seat on the handle (Fig. 8).

Fig. 7

Fig. 8

Transport and storage

In order to facilitate the transport of the rollator and to save space during its storage, front wheels can be turned over and the rear wheels holder can be removed (Fig9).

Fig. 9

Front wheels

To release the locking of the front wheels, the red button must be pressed (Fig. 10). Then the front wheels should be pulled forward and rotated 180° upwards. In order to be able to use the rollator again, the front wheels must be pulled forward and turned downwards. Locking means that the wheels are in the correct position, then the seat can be adjusted accordingly.

Fig. 10

Back wheels

The lever should then be folded (Fig. 11). In the Final step, the button is pressed and the rear wheels are pulled down. It should be noted that the back wheels are firmly connected to the roller conveyor via the brake line. To remove them, the rear wheel mechanism must be pushed back into the frame. Locking in means that the wheels are in the correct position, then the lever can be pushed back. Make sure the back wheels are properly fastened and that the brake is working properly.

Fig. 11

Walking with the rollator

First, test the use of the rollator on flat, dry ground. If possible, let yourself be accompanied. Practice with the rollator first, only when you feel safer can the walking aid be used on sloping surfaces.

When using the rollator, it should be remembered not to stand too far from the handles. Only in this way can a safe use of the walker and case safety be guaranteed. The bent posture is to be avoided.!

The tempo and the direction of travel is chosen independently by pushing the rollator in Front of you. To reduce the running speed, the brakes should be used. When braking, both brakes must always be applied, otherwise the rollator with the unbraked wheel can turn around the braked wheel and cause the user to fall.

Always try to bypass obstacles on your way. At steps you are looking for a ramp, at stairs you take the elevator. If this is not possible, the rollator must be carried over the obstacle. If possible, seek the help of a second person.

Tilting aids

I Your rollator is equipped with two tilting aids (Fig. 12) to help you overcome curbs. By stepping your foot on the tilting aid and at the same time attracting the rollator to you on the handles. lift the front wheels. You can overcome curbs and other minor obstacles in this way.

Please make sure that you are in a stable position.

Fig. 13

Brakes and parking brakes

I Your rollator is equipped with a manually operated service brake and a parking brake against unintentional rolling away.

To bring the rollator to a standstill easily and safely, pull both brake levers upwards at the same time (Fig. 13).

The braking effect depends on how you actuate the parking brake. Press down the brake levers until they are locked in with a clearly audible click (Fig. 14). To release the parking brake, pull the brake levers back up (Fig. 15).

Fig. 14

The rollator as a seat

Your rollator is equipped with a seat and a backrest. Before you sit down, it is important to put on the parking brake. The rollator might otherwise roll away, and you could fall. Do not use an extra seat cushion, it could slip away and cause your fall.

Fig. 15

Carrying the rollator

If you want to carry the rollator, e.B over a curb, fold it together. Wear the rollator on handles or on the frame. The risk of clamping is to be considered.

Safety devices

Before each use, the rollator must be inspected for damage and the brakes must be checked for their proper functioning. The rollator is not suitable for use on an escalator (danger of falling!). The rollator is made of non-combustible materials. The maximum user weight is 136 kg, the maximum payload of the carrying bag is 5 kg. The rollator must not be used to transport loads. The carrying bag is not designed for

transport of animals. The rollator can be folded in a space-saving manner. Please note that there is a risk of clamping when folding and folding. To fold or unfold the rollator, do not pull on the attachments (brake, wheels, bag, etc.) but always on the frame of the rollator. Please note that if the seat and the backrest are exposed to direct sunlight, they can heat up and cause the burning of the user's skin. Therefore, the seat and backrest or the entire rollator should be covered to protect them from the effects of the sun's rays.

Cleaning

The frame parts can be wiped with a damp cloth. For heavy dirt, you can use a mild detergent. The wheels can be cleaned with a damp brush with plastic bristles (do not use a wire brush!) The backrest, seat and transport bag can be washed off with a mild soapy liquor.

Do not use a high-pressure or steam cleaner! After cleaning, the rollator should be dried with a towel or left to dry.

Desinfektion

The rollator can be disinfected with a household disinfectant. Use approved spray disinfectants or wipe disinfectants on an alcoholic basis. The Robert Koch list of approved disinfectants can be found on http://www.rki.de.

After disinfection, the rollator should be left to dry in fresh air, taking into account the duration of action of the disinfectants specified by the producers.

Technical data

| Dimensions and weights | | | |
|---|--------------------|--|--|
| Seat width | 450 mm | | |
| Seat height | 540 mm | | |
| Overall width | 653 mm | | |
| Total height / height sliding terms (6-fold adjustable) | 810-936 mm | | |
| Kerb weight (without accessories) | 7,85 kg | | |
| Maximum load | 136 kg | | |
| Maximum load on the bag | 5 kg | | |
| Folding dimensions | 706 x 250 x 810 mm | | |

| Wheels (puncture proof) | | | | |
|-------------------------------|-------------------------------|--|--|--|
| Steering front wheels (front) | 250x35 mm | | | |
| Brake wheels, rigid (rear) | 250x35 mm | | | |
| Turning radius | 1770 mm | | | |
| Materials | | | | |
| Frame | Aluminium (pulverbeschichtet) | | | |
| Seat | Nylon, black | | | |
| Backrest | Nylon, black | | | |
| Wheels | Polyurethane | | | |
| | | | | |

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ANTAR Sp. J. 03-068 Warszawa, ul. Zawiślańska 43, tel.: +48 22 518 36 00, fax.: +48 22 518 36 30, e-mail: antar@antar.net